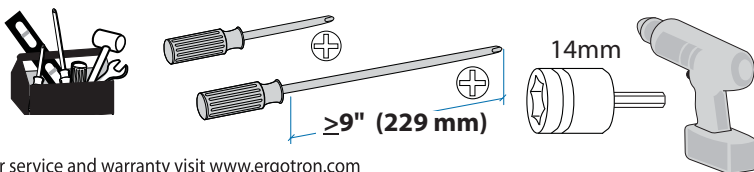
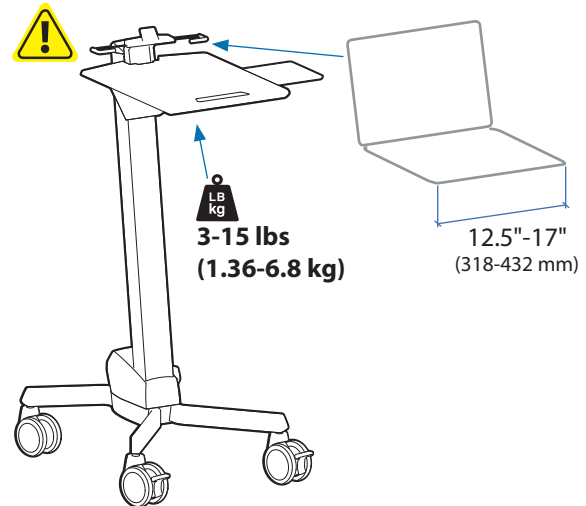
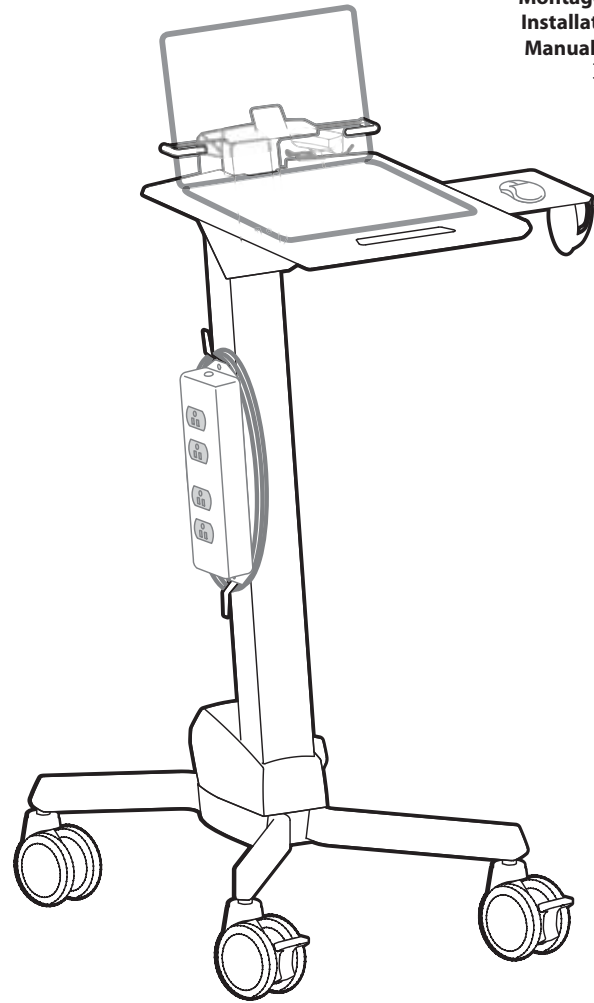
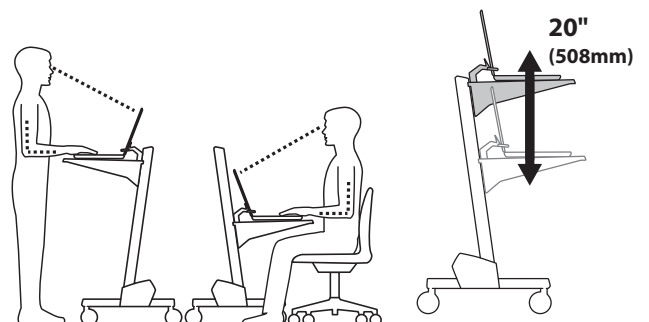
www.ergotron.com
 USA 1-800-888-8458
 Europe +31 (0)33-45 45 600
 China 86-769-86018920



	A	B	C
1	1x 	1x 	4x
2		1x 	4x
3		1x 	2x
4		1x 	Security Screw Tornillo de seguridad Vis de sécurité Sicherheitsschraube mit Innenfünfkant Veiligheidsleutel Vite di sicurezza 固定ネジ 安全螺丝 Security Screw(안전 스크루)
5		1x 	
6	1x 		5-sided Security Wrench Llave de seguridad pentagonal Clef Allen de sécurité Fünfkant-Inbusschlüssel 5-kantige veiligheidsleutel Chiave pentagonale di sicurezza 5角固定用レンチ 5角安全扳手 5면 Security Wrench(안전 렌치)
7			
8	4x 		
9	2x 		1x
10	4x 	2x 	1x
11	4x 	8x 	4x


For service and warranty visit www.ergotron.com
 Si desea información de servicio y garantía, visite www.ergotron.com
 Pour toute question concernant la garantie ou le service client, consultez www.ergotron.com
 Weitere Informationen zu Service und Gewährleistung erhalten Sie unter www.ergotron.com.
 Ga voor service en garantie naar www.ergotron.com
 Per assistenza e la garanzia visitare www.ergotron.com
 Gå in på www.ergotron.com för service och garanti
 サービスおよび保証については、www.ergotron.com をご覧ください。
 有关服务和保修，请访问 www.ergotron.com





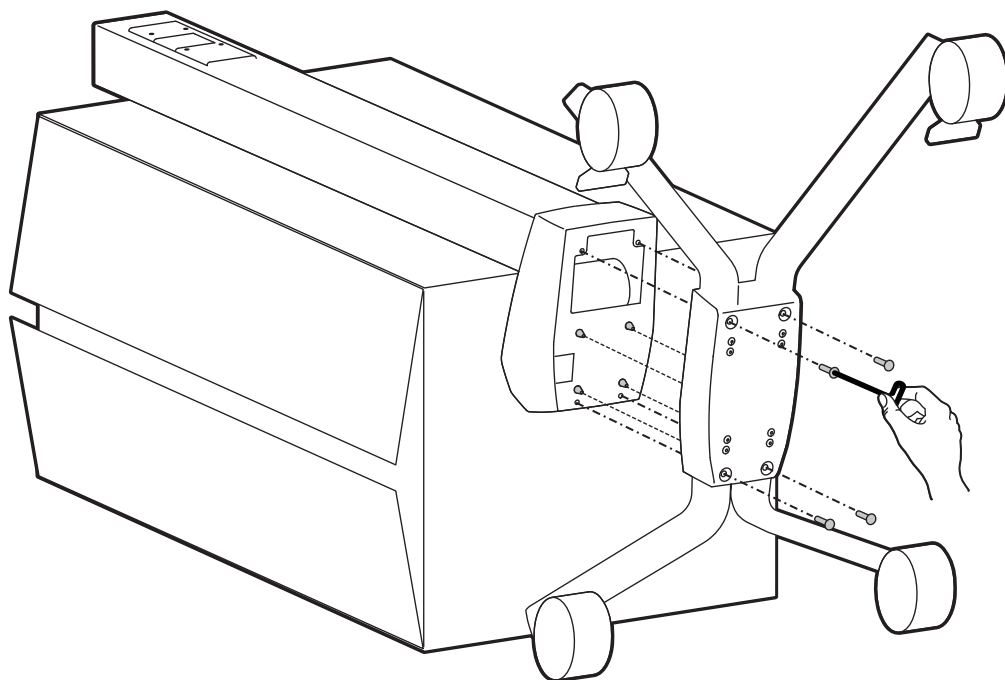
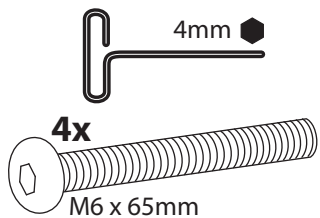
1





2

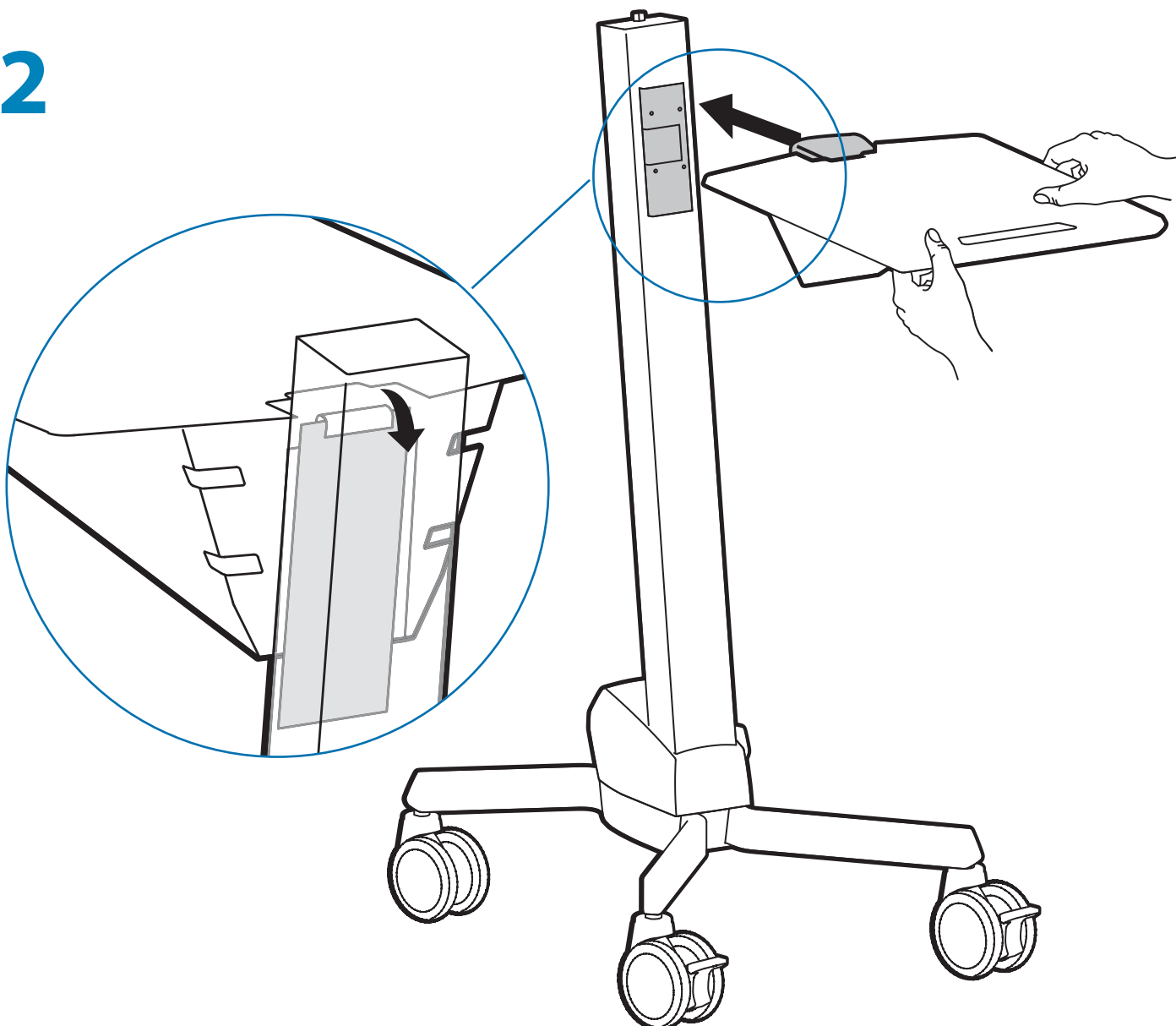
3

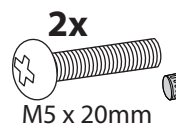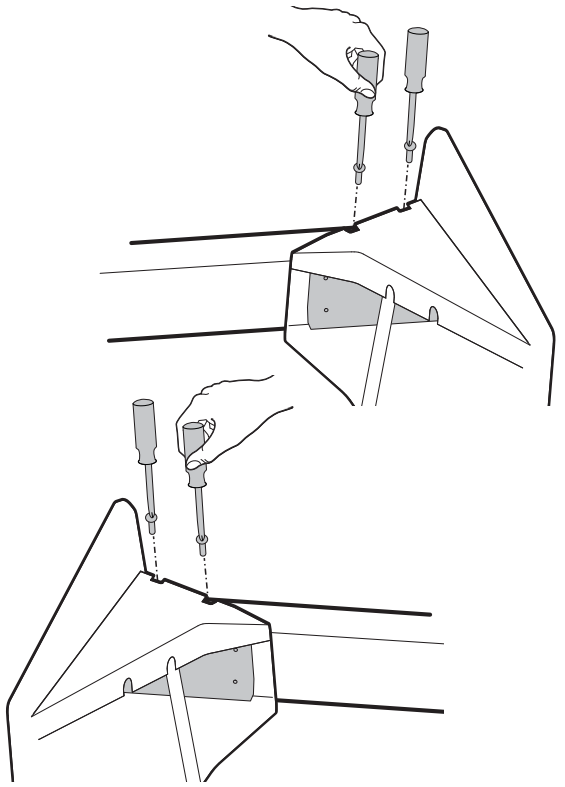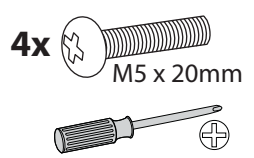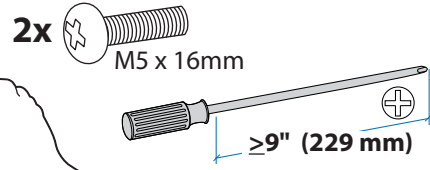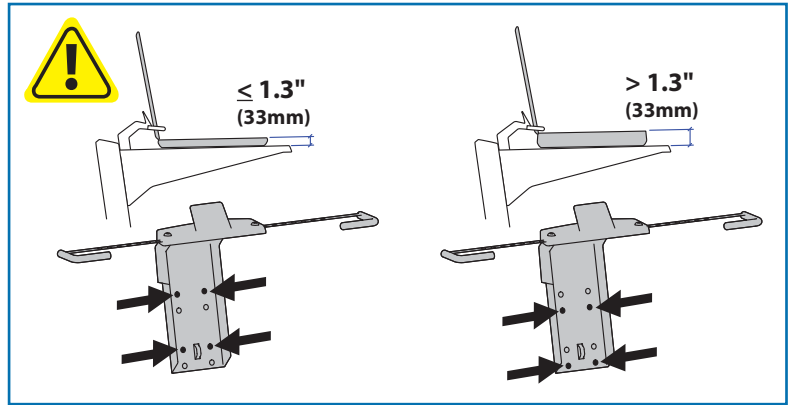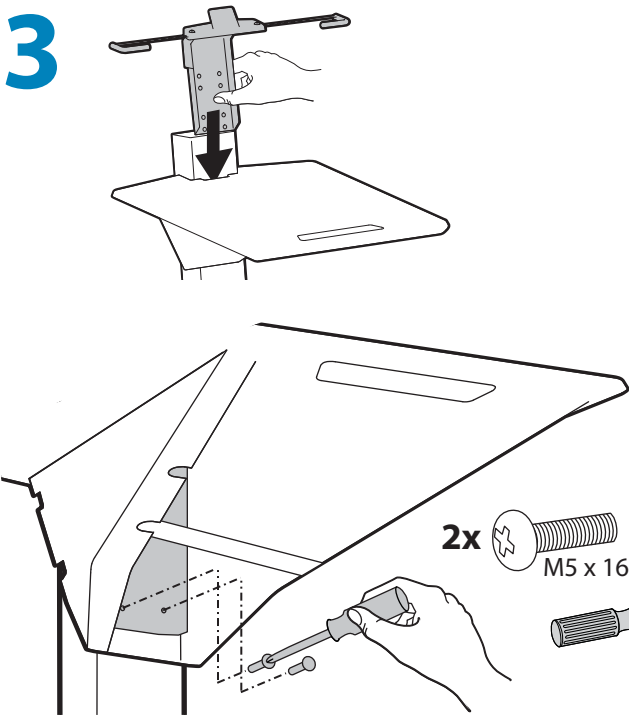OPTIONAL: Security Screw. 5-sided Security Wrench
 Opcional: Tornillo de seguridad. Llave de seguridad pentagonal
 Facultatif: Vis de sécurité. Clef Allen de sécurité
 Optional: Sicherheitsschraube mit Innenfünfkant. Fünfkant-Inbusschlüssel
 Optioneel: Veiligheidssleutel. 5-kantige veiligheidsleutel
 Opzionale: Vite di sicurezza. Chiave pentagonale di sicurezza
 オプション: 固定ネジ. 5角固定用レンチ
 可选: 安全螺丝. 5角安全扳手

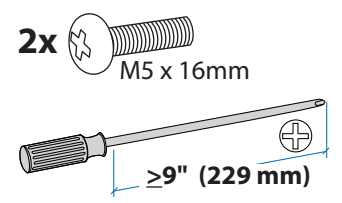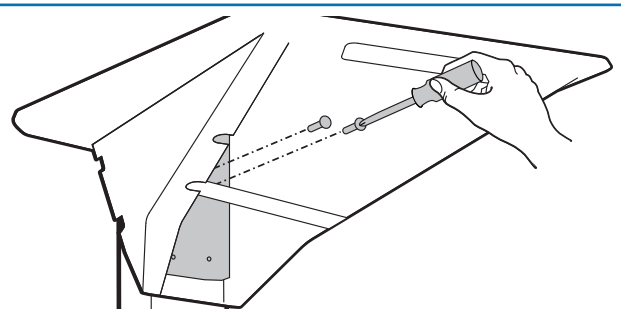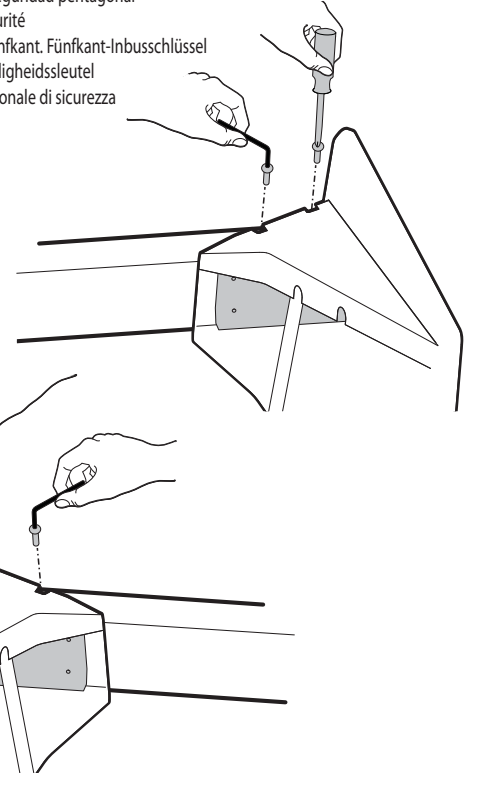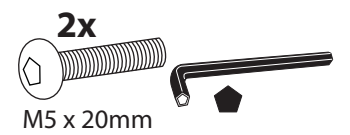
4

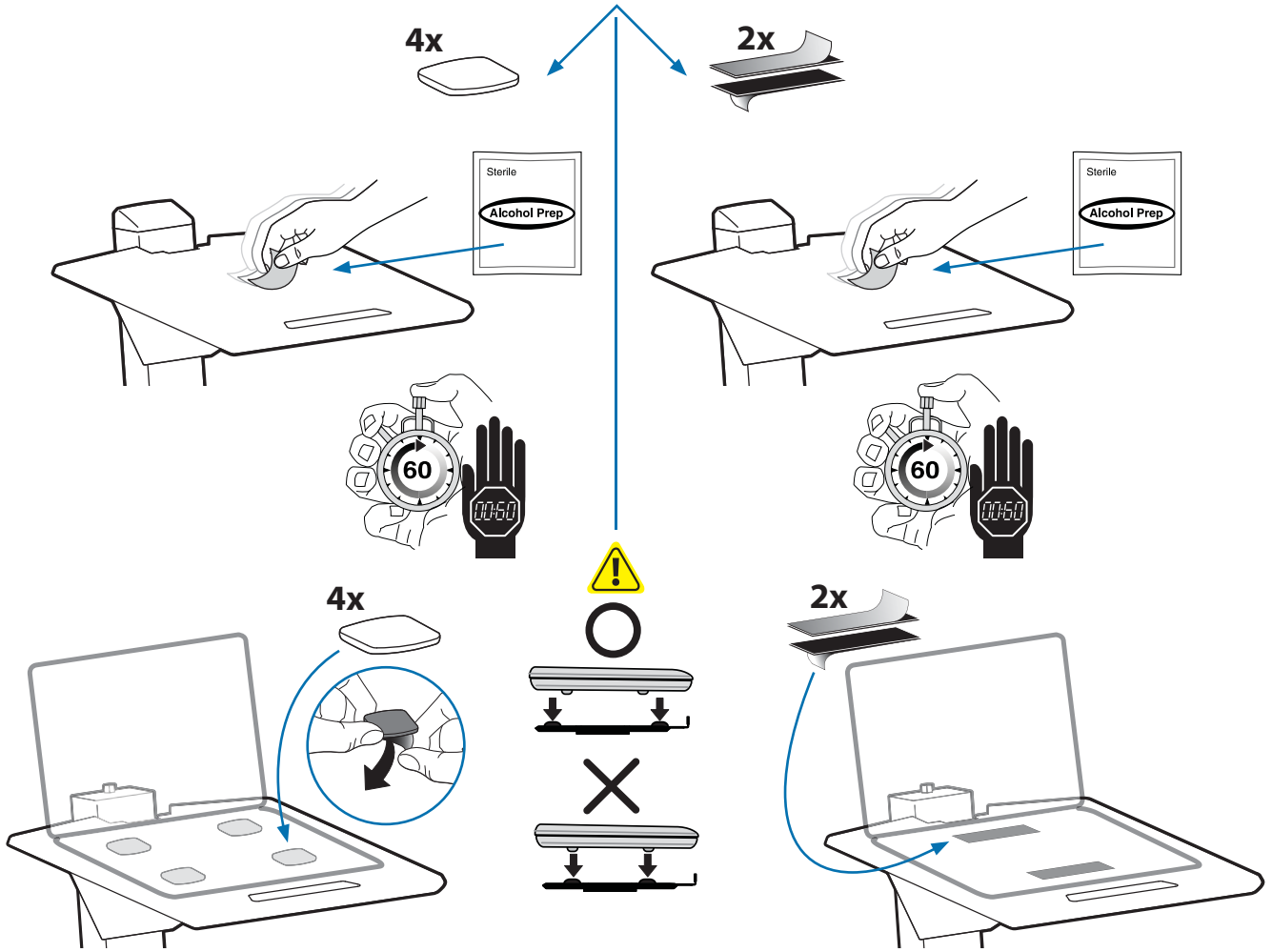
5

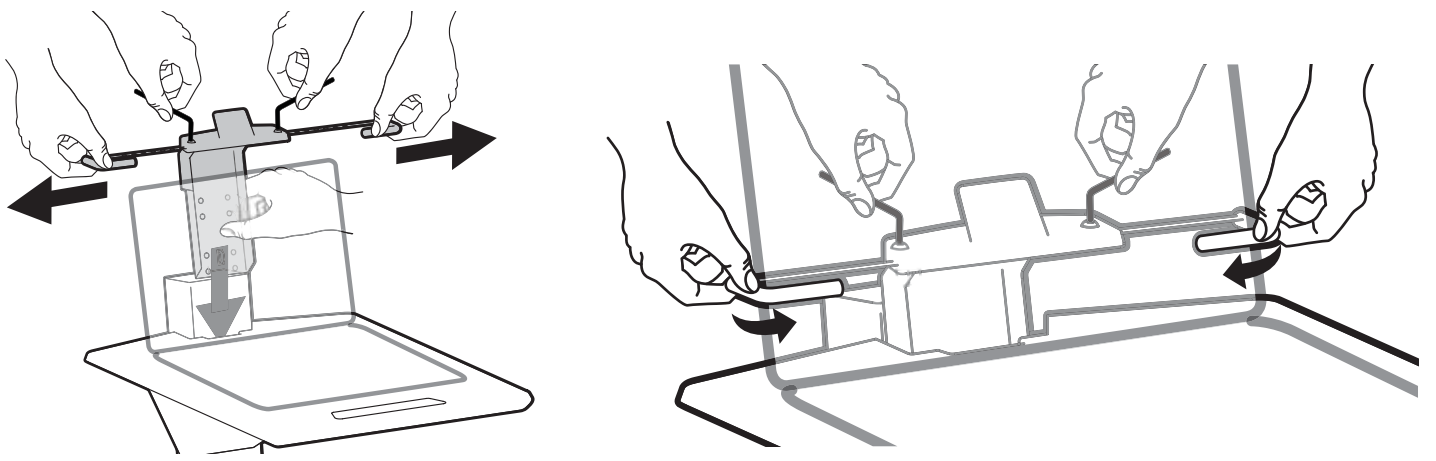
5-sided Security Wrench
 Llave de seguridad pentagonal
 Clef Allen de sécurité
 Fünfkant-Inbusschlüssel
 5-kantige veiligheidsleutel
 Chiave pentagonale di sicurezza
 5角固定用レンチ
 5角安全扳手
 5면 Security Wrench(안전 렌치)

6

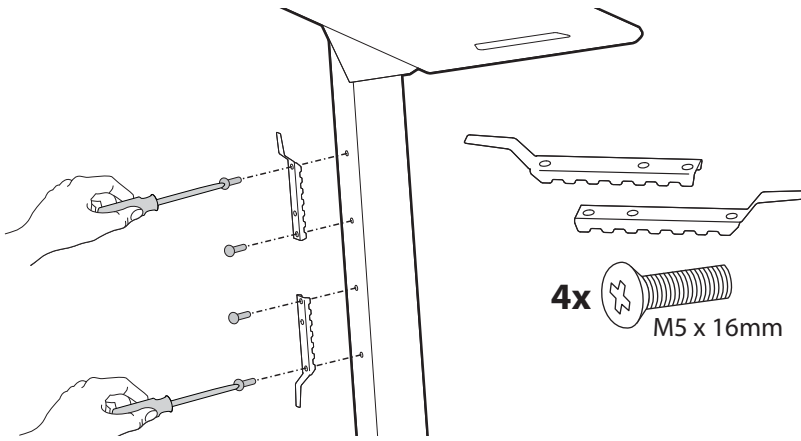
4x  M5 x 16mm

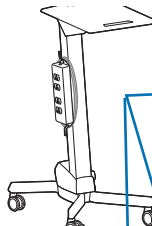Mounting Holes
Orificios de montaje
Trous de montage
Befestigungslöcher
Montageopeningen
Fori di montaggio
マウントホール
安装孔


2x  M4

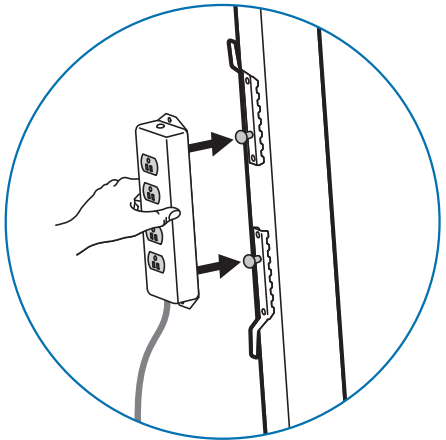
Fasteners supplied with Ergotron Medical-Grade Power Strip.
 La regleta de enchufes de calidad médica de Ergotron incluye abrazaderas.
 Fixations fournies avec le câble d'alimentation de niveau médical d'Ergotron.
 Im Lieferumfang der Ergotron-Steckdosenleiste für den medizinischen Gebrauch enthaltene Befestigungsmittel.
 Bij de Medical Grade Power Strip van Ergotron geleverde bevestigingsmaterialen.
 Dispositivi di fissaggio forniti con la presa multipla per apparecchi medicali Ergotron.
 エルゴトロン医療レベル パワー ストリップに付属のファスナー。
 Ergotron 医用電源插板附带的紧固件。


Fasteners supplied with Ergotron Standard 6 Outlet Power Strip.
 La regleta de 6 enchufes de calidad estándar de Ergotron incluye abrazaderas.
 Fixations fournies avec le câble d'alimentation 6 prises standard d'Ergotron.
 Im Lieferumfang der Standard-Ergotron-6-Steckdosenleiste enthaltene Befestigungsmittel.
 Bij de Standard 6 Outlet Power Strip van Ergotron geleverde bevestigingsmaterialen.
 Dispositivi di fissaggio forniti con la presa multipla standard (6 prese) Ergotron.
 エルゴトロン標準6コンセント パワー ストリップに付属のファスナー。
 Ergotron 标准 6 插口电源插板附带的紧固件。

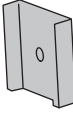2x  M4

2x 

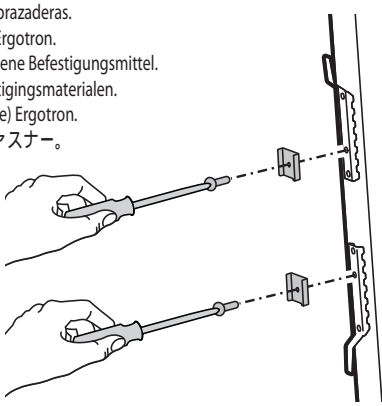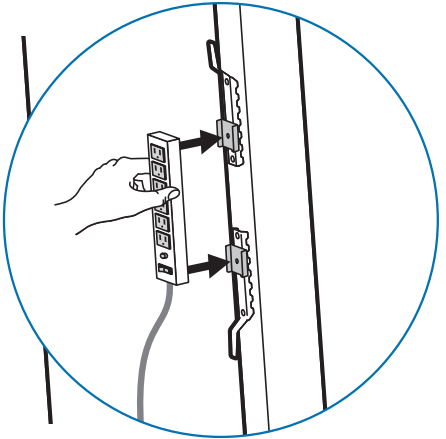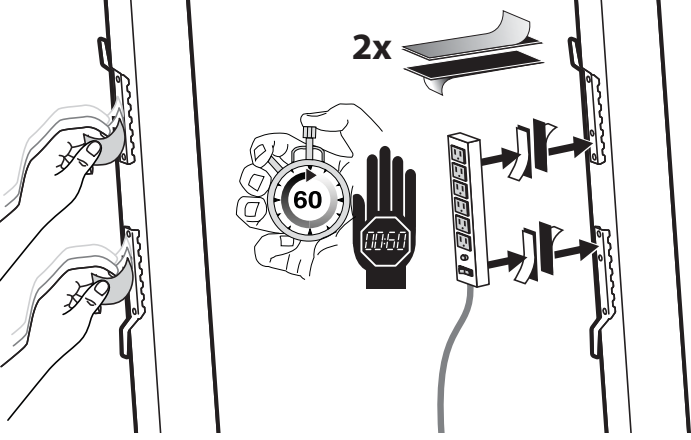
2x 

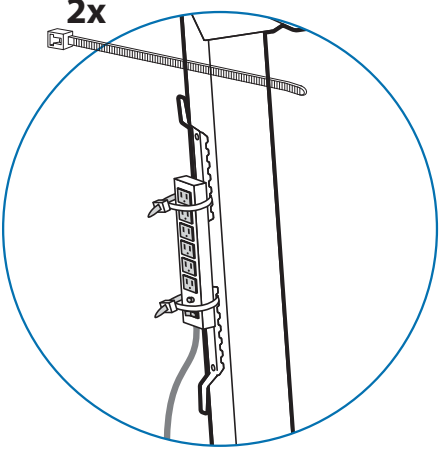
2x 




2x 


7

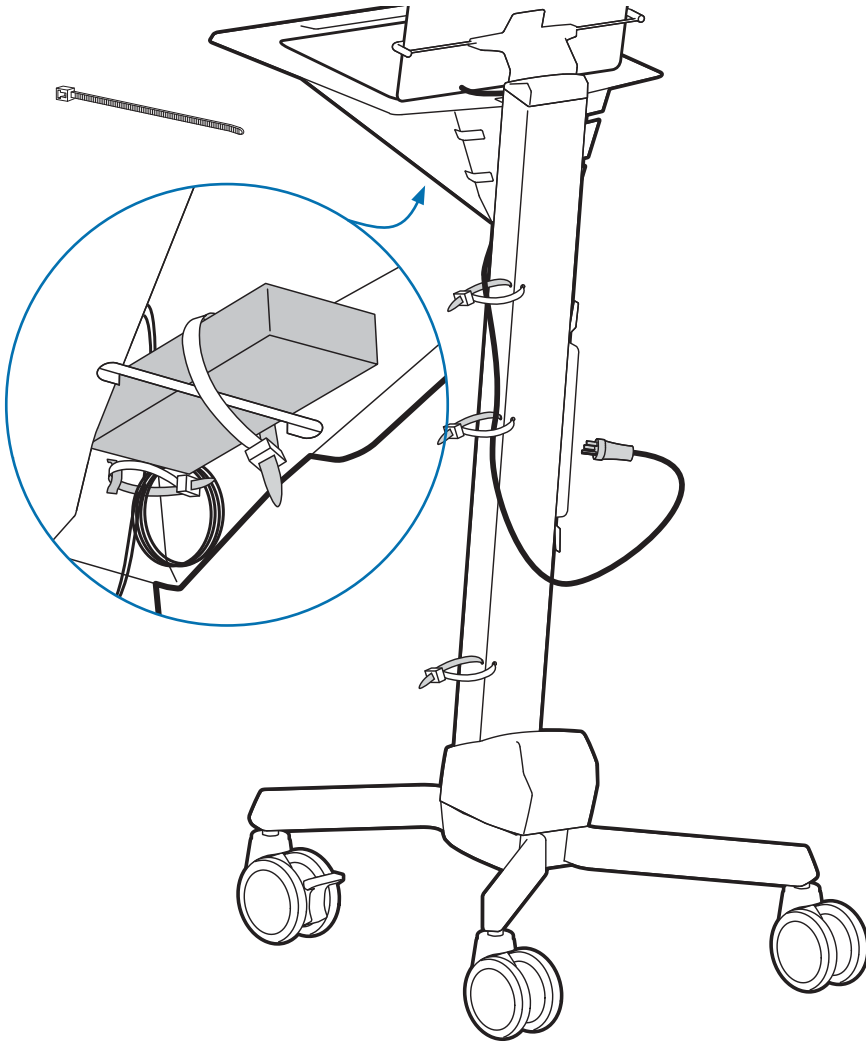8

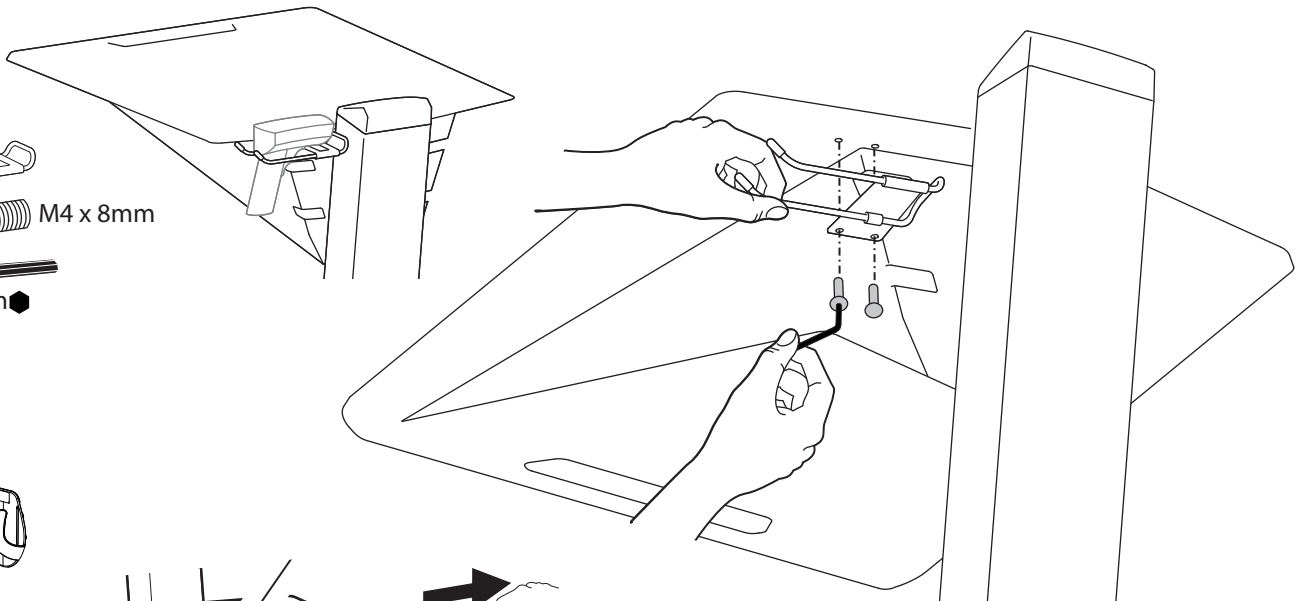
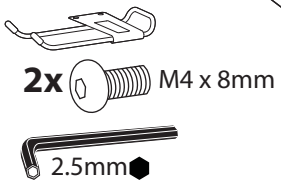
9

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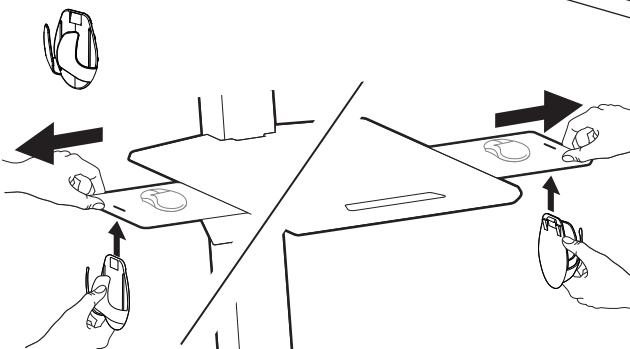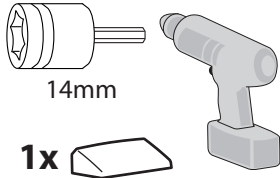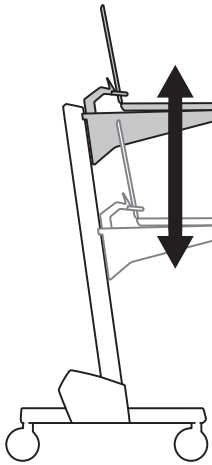WARNING

Impact Hazard!
Moving Parts can Crush and Cut.
 Raise to the full up position before removing equipment.
 Always adjust the lift tension to accommodate applied load.
 DO NOT add weight below minimum or above maximum weight capacities.
Failure to heed this warning may result in serious personal injury or property damage.

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© 2010 Ergotron, Inc.
 All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.